

# BPSim™ Getting Started

## BPSim™ Application

### Download the BPSim™ application.

Download the BPSim™ app via the Google Play Store (Android) or via the App Store (iOS). Utilize your BPSim™ User Guide (on included flash drive) to set up the Wi-Fi. The BPSim™ User Guide can also be found at: [simplebooklet.com/bpsimguide](http://simplebooklet.com/bpsimguide).

## BPSim™ Device

### Fully charge your BPSim™ before use.

The device was fully charged upon leaving our facility, however due to shipping times the battery may drain. A fully charged battery should last 6-8 hours.

**Device is fully charged when LED indicator light on charger turns green.**

### Remove BPSim™ on/off mode adapter to use.

The on/off adapter *does not* charge the device, but enables device Wi-Fi and provides power to the device when removed.

### Replace BPSim™ on/off mode adapter when not in use.


The on/off adapter *should always* be put back on the device when not in use. This adapter preserves the BPSim™ battery when the device is not in use. Please put the on/off adapter back for periods in which the BPSim™ is not in use or charging.

## CHARGER



- CUFF DISCONNECTED
- CHARGING
- CHARGE COMPLETE

## OFF-MODE

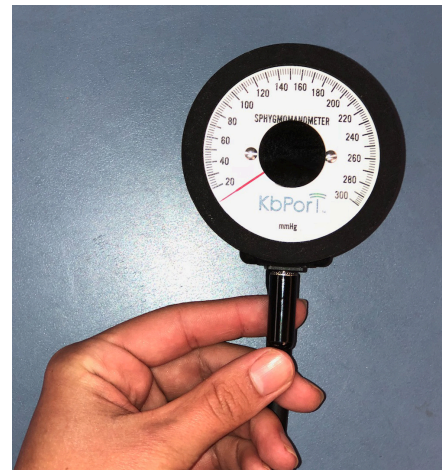

## ON-MODE

